




Refer to User Manual for comprehensive information

Control Panel

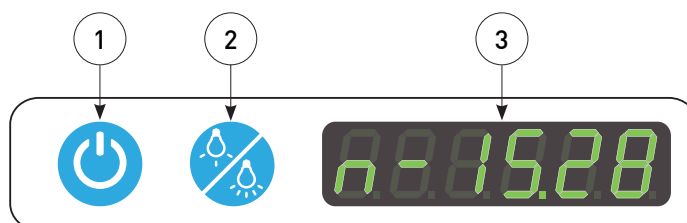

1	On/Off Button	Press this button to turn Colibri Phototherapy display and functions on and off.
2	Therapy Mode Button	Press this button to switch between standard and intensive therapy mode.
3	Display	Shows therapy mode and treatment time.

Selecting therapy mode

Colibri Phototherapy can be operated in 2 modes:

- STANDARD: ~33 $\mu\text{W}/\text{cm}^2/\text{nm}$
- INTENSIVE: ~57 $\mu\text{W}/\text{cm}^2/\text{nm}$



Turn on the device by pressing the "On/Off Button".
 The display shows the "total device hours".
 This will be displayed for 3 seconds.



The display will start counting the "treatment time" in STANDARD MODE.



To select INTENSIVE mode press the "Therapy Mode Button".
 The display shows the "total device hours of INTENSIVE MODE".
 This will be displayed for 3 seconds.




The display will start counting the "treatment time" in INTENSIVE MODE.



Turn off the device by pressing the "On/Off Button".

Irradiance

STANDARD MODE

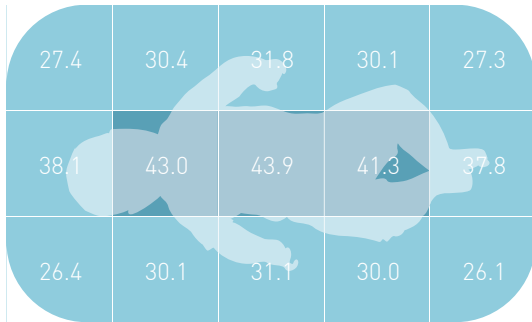

INTENSIVE MODE



IRRADIANCE

$\mu\text{W}/\text{cm}^2/\text{nm}$

20.0 - 39.9

40.0 - 59.9

60.0 - 79.9

Height adjustment



Recommended distance from the light source to the patient is 30-40 cm.



Exercise caution when adjusting the height of the Colibri Phototherapy.