






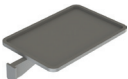







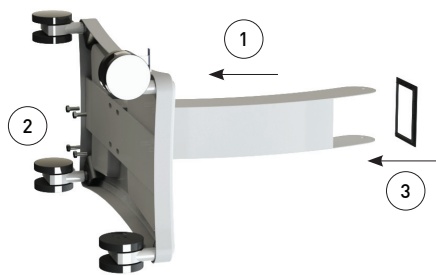
 Refer to User Manual for comprehensive information:
<http://www.mttts-asia.com/wallaby-warmer/>

STEP 1 - Remove components from the box

- | | | | | | |
|---|-----|---------------------|--|-----|-----------------------|
|  | x 1 | Base |  | x 2 | Side Panel 54cm |
|  | x 1 | Stand |  | x 2 | Side Panel 64cm |
|  | x 1 | Stand plastic frame |  | x 1 | Bed Mattress |
|  | x 1 | Control unit |  | x 2 | Storage Shelves |
|  | x 1 | Bed with X-ray tray |  | x 4 | Bolt M10x80 |
| | | |  | x 4 | Bolt M10x25 |
| | | |  | x 1 | Bolt M10x120 + 2 Nuts |
| | | |  | x 1 | Bolt M10x60 + 1 Nut |
| | | |  | x 2 | Skin sensor |

STEP 2 - Attach Stand to Base


 Two people might be required to perform this step




STEP 3 - Attach Control Unit to Stand




STEP 4 - Attach Side Panel to Bed

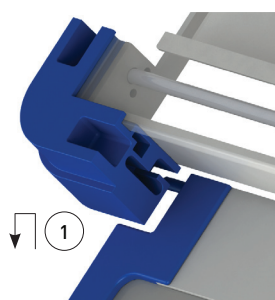


 Attach only side panel that is closest to the Control Unit in this step. Remaining panels will be attached in Step 6.


STEP 5 - Attach Bed

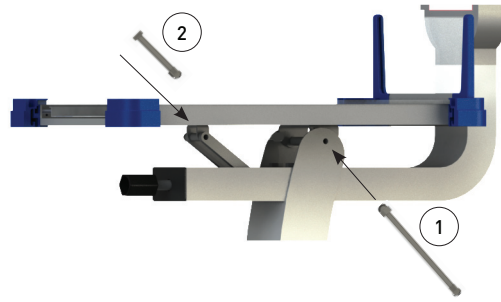

STEP 6 - Attach 3 remaining Side Panels to Bed

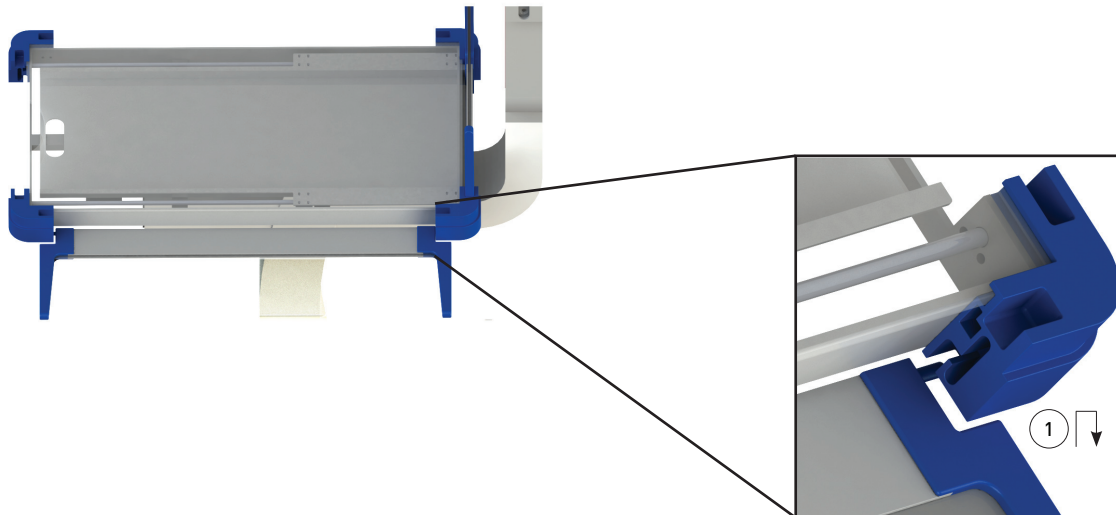

STEP 8 - Attach Storage Shelves

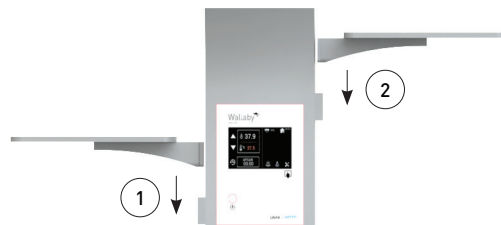

STEP 9 - Plug in and power up

