



Refer to User Manual for comprehensive information:
<http://www.mttts-asia.com/wallaby-warmer/>

Optional Components List



IV Pole (WLB-1030)



Storage Drawer (WLB-1050)



Storage Shelf (WLB-1040)



Cylinders Holder (WLB-1060)


Cylinders Holder (WLB-1060)



2 x Bolt M10x25

Storage Drawer (WLB-1050)

- ① Disconnect top and bottom parts
Press black latch on both sides and slide out

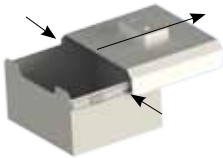

- ② Attach the top part to the control unit
Use 2 x Bolt M10x25



- ③ Connect the bottom part
Slide in until you hear a 'click' sound



IV Pole (WLB-1030) and Storage Shelf (WLB-1040)

