



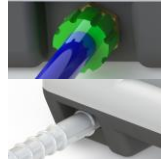

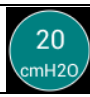
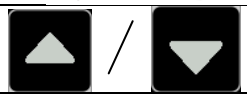

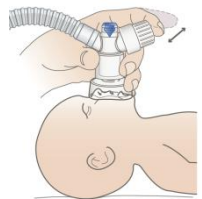


 Refer to User Manual for comprehensive information:
<https://www.mttts-asia.com/beluga-resuscitator/>

SETUP

- 1 Press and hold the power button to turn the device ON/OFF 
- 2 Check battery status, connect 12V adapter if it's not fully charged. 
- 3 Calibrate oxygen sensor  → 
- 4 Connect oxygen source and T-piece circuit
Caution: If a flow meter is attached, it should be fully open. 

OPERATION

- 5 Occlude PEEP cap. Leave blue cap in place on the T-piece circuit or connect test lung to the T-piece circuit. 
- 6 Select PIP on screen  → 
- 7 Set PEEP by adjusting PEEP cap and observe measured pressure on the screen.
Caution: Always readjust PEEP after changing PIP. 
- 8 Double check PIP/PEEP by placing and removing thumb over the PEEP cap. 
- 9 Fit T-Piece to patient mask and start resuscitation by placing and removing thumb over the PEEP cap. 
- 10 Adjust desired FiO2 by selecting on the screen.  → 